

Aluminum Series Back Supports Loop Handle Installation Guide

STEP 1



With cushion removed, insert M6x10mm button head with one serrated washer through the inside of the backrest.

STEP 2



After the bolt is through the AL shell, place an additional serrated nut on bolt on the outside of the shell

STEP 3



With the cable fully inserted into the cable end piece, thread bolt into cable end.

Note: The M6x10mm bolt will crimp the cable inside the cable end piece, securing cable in place.

STEP 4



Repeat steps 1-3 for the other mounting hole.

Note: The cable should exit the cable ends on top.



Note: The black cable ends should be tight against the shell, as well as the cable crimped tightly inside the cable ends. Pull on the cable to ensure everything is secured properly.